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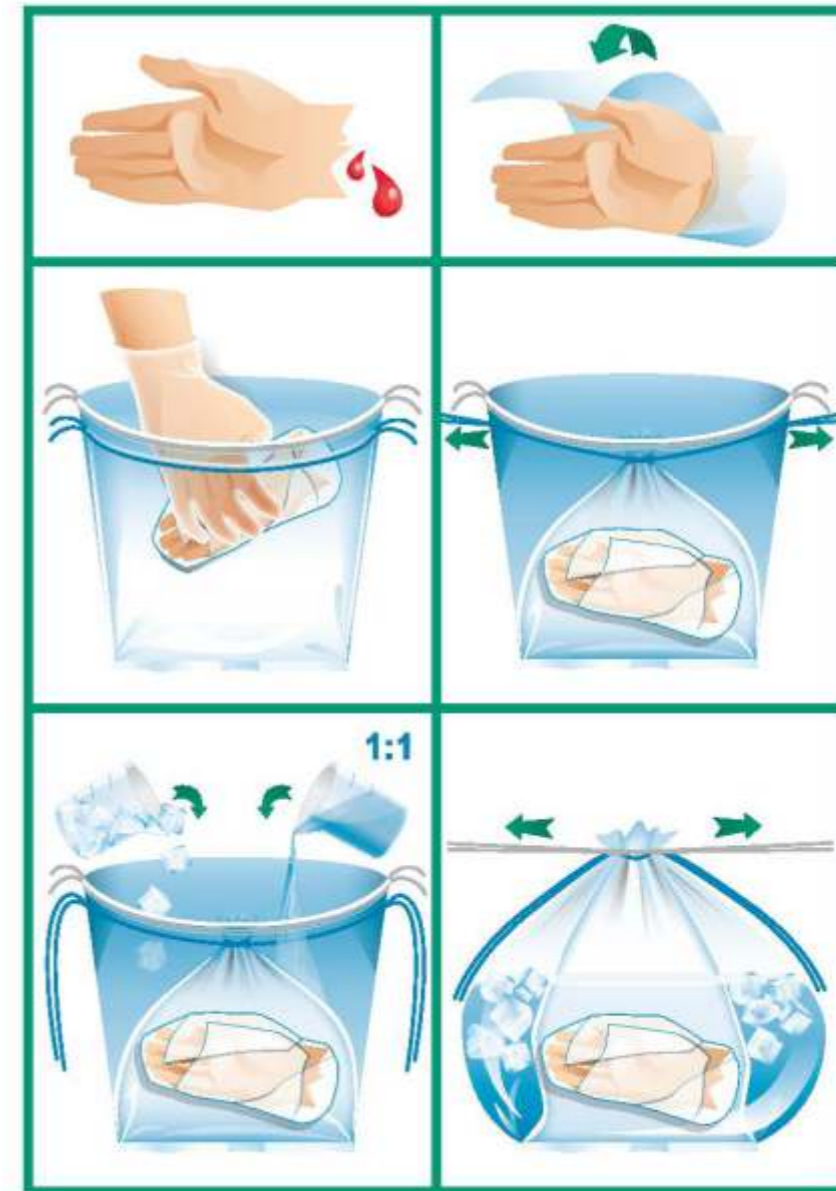
REF CPC0001/220450

EN LIMB TRANSPORT BAG SIZE M

A limb transport bag, suitable to hold and carry amputated limbs. The system requires the use of two bags (one inside the other), both of them are reclosable. The amputated limb is put into the inner bag, which is cooled by the action of the outer one.

How to use:

Wrap the limb in a sterile gauze, place it into the inner bag and make sure that it has been closed properly. Activate the cold packs and place them into the outer bag, making sure to surround the limb contained into the inner bag. Protect the bag from shocks and make sure that the cold packs work properly.

Warnings:

The use is allowed only to a qualified person.

Keep the device away from direct sunlight and protect it from heat and moisture.

Do not use if the package is damaged. Visually inspect for damages or breaches of packaging integrity prior to use.

For single use only! Reusing a single-use medical device is dangerous. Reprocessing devices, in order to reuse them, may seriously damage their integrity and their performance.

Products shall not be re-sterilized.

Serious incident reporting:

For a patient/user/third party in the European Union and in countries with identical regulatory regime (Regulation (EU) 2017/745 on Medical Devices); if, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and/or its authorized representative and to your national authority.

Product disposal:

In order to minimize the risk of potential infection hazards, or environmental pollution, disposable components of the medical device should follow disposal procedures according to *applicable* and local laws, rules, regulations and infection prevention standards.

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WERO GmbH & Co KG
CH REP Swiss AR Services GmbH