

TOBIN®

Eyewash



MD

STERILE



REF 78

CE 0402



1

- Begin washing the eye immediately.
- Damage increases by the second.
- Always wash under the eyelids.
- Interrupt washing to allow blinking and eye movements.
- Washing periods only apply when MOST of the liquid reaches the eye.
- Control the flow by covering the bottom hole with the thumb.



2

Mechanical Damage

5-15 minutes

If required, see a doctor

Alkalis

10-20 minutes

Continue washing on the way to a doctor



3

Acids

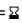
5-15 minutes

See a doctor

Manufactured by:

TOBIN 
SWEDEN AB

Arntorpsgatan 26
442 45 Kungälv
Sweden
www.tobinsweden.se

Contents: 1 litre Sterile Saline Solution 0.9%. Disposable, single use bottle. Empties in 3 min. under natural pressure. Unopened bottle has a shelf life of 3 years – see expiry date = 
Store at 2-35 °C.
Do not place in direct sunlight.