


## FAST-LIFT X



## FAST-LIFT

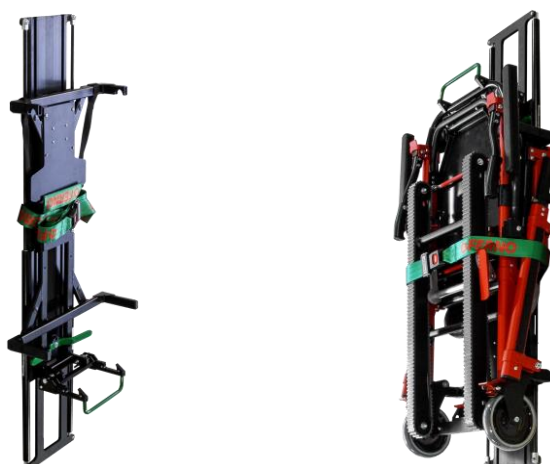



*TIP: Regular cleaning helps reduce the risk of transmitting disease and enables the equipment to function at its optimum.*

## Disclaimer

This manual contains general instructions for the use, operation, and care of the FAST-LIFT, FAST-LIFT X. Safe and proper use of the FAST-LIFT, FAST-LIFT X is the sole responsibility of the user. Safety information is provided as a service to the user. All other safety measures should comply with applicable regulations. It is recommended that users receive training on the proper use of the FAST-LIFT, FAST-LIFT X before deploying it in an actual situation. Retain this manual for future reference.

## **FAST-LIFT, FAST-LIFT X** **Product Specification**

<b>Length:</b>	1296 mm
<b>Width:</b>	468 mm
<b>Depth:</b>	FAST-LIFT X - 283 mm / FAST-LIFT - 212 mm
<b>Weight:</b>	FAST-LIFT X - 19 kg / FAST-LIFT - 10,5 kg
<b>Load Limit:</b>	30 kg

<b>Product Number</b>	<b>Product Name</b>
60-0254-001	FAST-LIFT X
60-0254-002	FAST-LIFT



## FAST-LIFT X Features

- Is suitable for FAST PREMIUM, FAST, and FST chairs
- Provides assisted hydraulic lifting (for example onto ambulance wall or doors, possibly also into recessed storage spaces)
- With adjustable forward/backward extension
- Safely secures the chair


## Procedure for Removing the Chair from the Adapter



Ensure that FAST-LIFT X is in the standard position (Figure 1).



Unfasten the restraint and pull the FAST-LIFT X towards you while unlocking the green lever located on the side of the FAST-LIFT X (Figure 2).

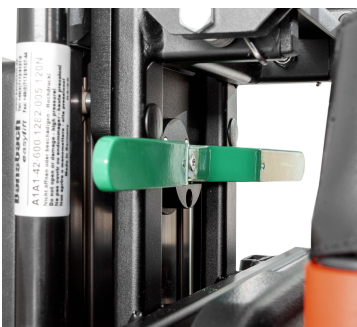

Pull the green lever towards you and move the FAST-LIFT X downward. Multiple locking positions are available (Figure 3).



Unlock the chair using the green lever located on the top side of the FAST-LIFT X. Pull the chair out for use. Follow steps 4 to 1 in reverse order (Figure 4).




## Procedure for Installing the Chair onto the FAST-LIFT X




Ensure that the FAST-LIFT X is in the standard position (Figure 1).



Unfasten the restraint and pull the FAST-LIFT X towards you while unlocking the green lever located on the side of the FAST-LIFT X (Figure 2).

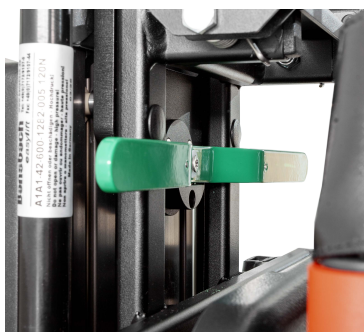


Pull the green lever towards you and move the FAST-LIFT X downward. Multiple locking positions are available (Figure 3).

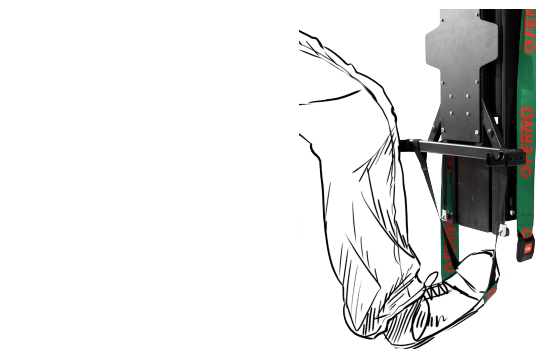


Pull the FAST-LIFT X down to the desired position; if needed, you may assist by using your foot placed in the strap. Before loading the chair, return the strap to its original position (Figure 4).



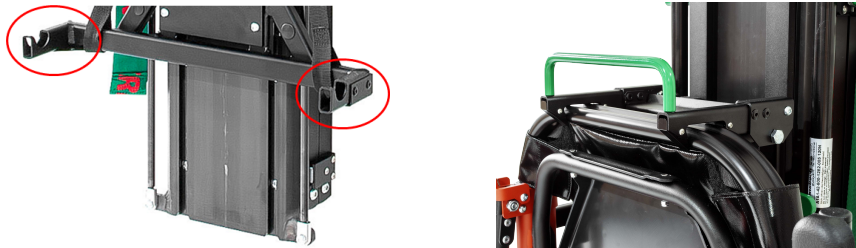

Place the chair onto the FAST-LIFT X. The lower part of the FAST-LIFT X is inserted into the lock, and in the upper position it clicks into place. Then, using the green lever located on the side of the FAST-LIFT X, raise the chair upward and press the chair with the FAST-LIFT X into position (Figure 5).

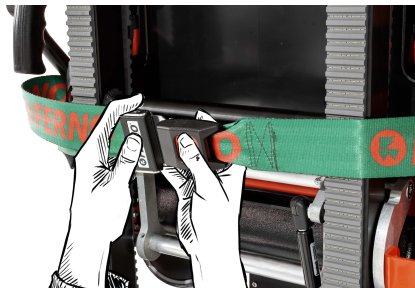

Secure the chair using the restraint (Figure 6).

## FAST-LIFT Features

- Is suitable for FAST PREMIUM, FAST, and FST chairs
- Provides assisted hydraulic lifting (for example onto ambulance wall or door)
- Safely secures the chair

Follow the same procedure for the FAST-LIFT in vertical direction, except horizontal direction.

### *[Cleaning and Disinfection]*

The LAST-LIFT, FAST-LIFT X is easy to clean and disinfect. Wipe all surfaces thoroughly with a suitable disinfectant. Follow the disinfectant manufacturer's instructions regarding application method and contact time.

#### **Ferno Slovakia Contact:**

Ferno Slovakia s.r.o. Bošáca 893  
913 07 Slovakia

More information at: [www.fernosk.sk](http://www.fernosk.sk)

