


## FAST BIG WHEELS



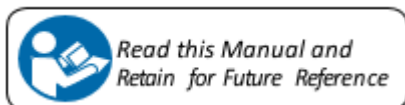

**TIP: Regular cleaning helps reduce the risk of transmitting disease and enables the equipment to function at its optimum**

## Disclaimer

This manual contains general instructions for the use, operation and care of this Fast Big Wheels. Safe and proper use of the Fast Big Wheels is solely at the discretion of the user. Safety information is included as a service to the user. All other safety measures taken by the user should be within and under consideration of applicable regulations. It is recommended that training on the proper use of this Fast Big Wheels be provided before using this Fast Big Wheels in an actual situation. Retain this manual for future reference.

## Fast Big Wheels Product Specification

**Wheel diameter:** 530 x 50 mm  
**Overall Dimension:** 660 x 560 x 530 mm  
**Weight:** 8 kg  
**Load Limit:** 250 kg

Product Number	Product Name
65-0407-001-P	Fast Big Wheels



## FAST BIG WHEELS Features

- FAST BIG WHEELS are an accessories for FAST Chair
- Suitable for off-road use
- Can be quickly and easily attached

## FAST BIG WHEELS Instructions for Use

To attach the wheels, press the button in the center of the wheel and insert the center pin of the wheel into the axle of the frame.



To detach the wheels, press the button in the center of the wheel and pull the wheel toward the operator.

### To attach the wheels frame:

Lift and place the rear axle of the chair into the U-bracket. Next, swing the top of the frame toward the chair until it locks into place. You should hear a click. The wheels are now secure.



**To detach the wheels frame:**

Push down on the bar marked "push to unlock" and allow the frame to swing down to the floor. Next, lift the rear axle of the chair out of the frames U-bracket. The frame is now detached.

### [Cleaning and Disinfection]

The Fast Big Wheels are easy to clean and disinfect. Wipe all surfaces with disinfectant. Follow the disinfectant manufacturer's instructions for application method and contact time.

**Ferno Slovakia Contact:**

Ferno Slovakia s.r.o. Bošáca 893


913 07 Slovakia

Tel: + 421 (0) 32 7708 010

More information at: [www.fernosk.sk](http://www.fernosk.sk)

**Warning**

**Please use under the guidance of the doctor or nurse**

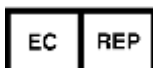

Ferno Washington Italia S.r.l.  
via B.Zallone, n26, 40066  
Pieve di Cento,  
Bologna, Italy  
+39.051.6860028