Model 71 Basket Stretchers



Model 71

Model 71-S

	71	71-S
A	86 in (2180 mm)	86 in (2180 mm)
B	24 in (610 mm)	24 in (610 mm)
C	8 in (190 mm)	8 in (190 mm)
	23 lb (10 kg)	24 lb (11 kg)
	600 lb / 272 kg / 42.9 st	

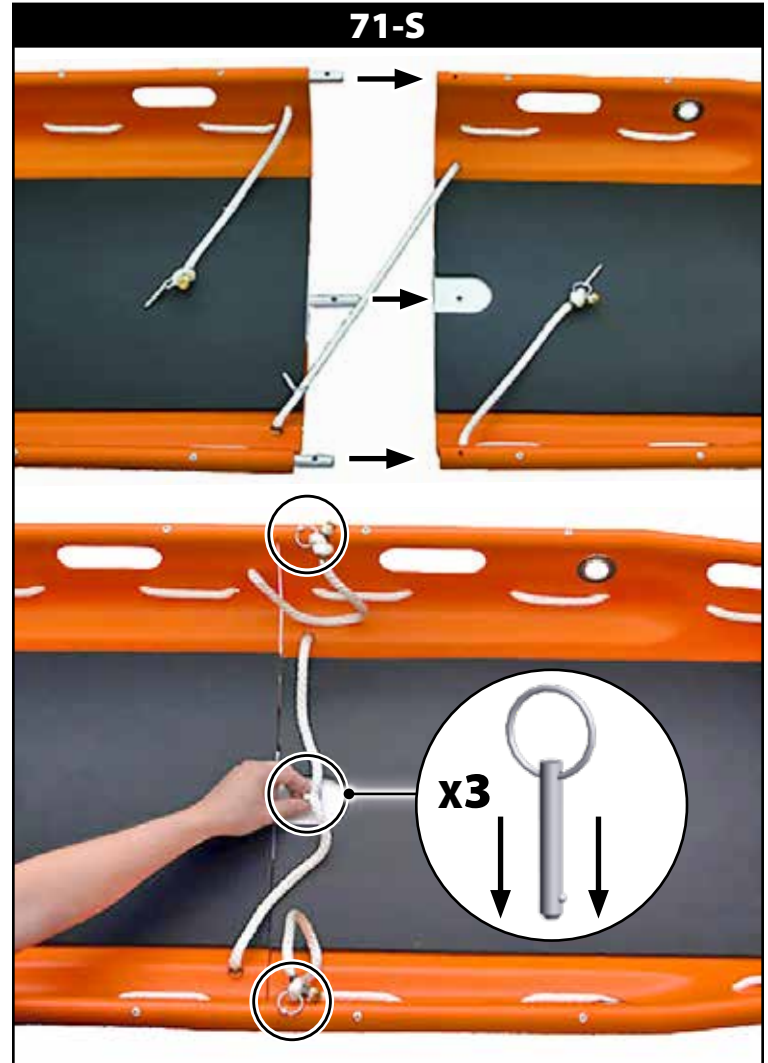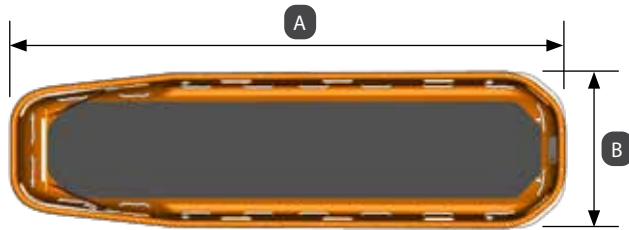Ferno-Washington, Inc.
70 Weil Way
Wilmington, OH 45177-9371, U.S.A.
1.937.382.1451

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




Ferno (UK) Ltd.
Ferno House, Stubs Beck Lane, Cleckheaton,
West Yorkshire, BD19 4TZ, UK
+44 (0) 1274 851 999

Model 71 Basket Stretchers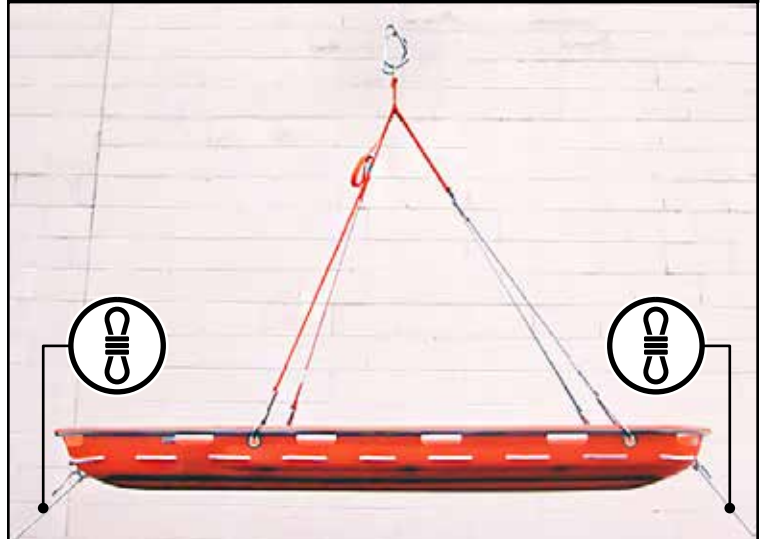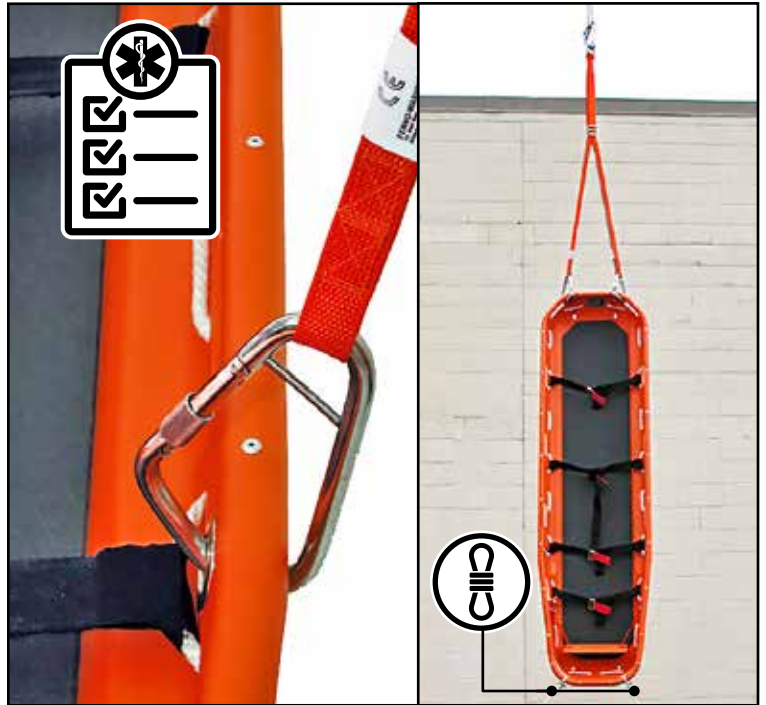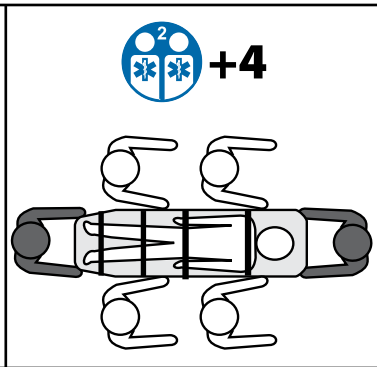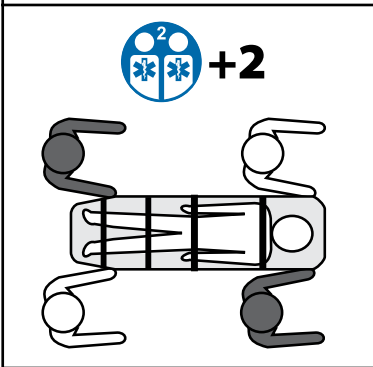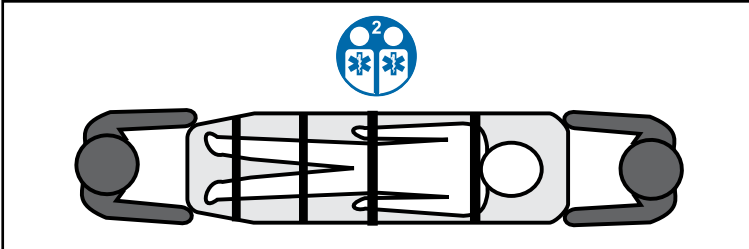

Description

The Model 71 Series Basket Stretchers are emergency patient-handling devices designed to transport patients to conventional cots or stretchers from areas that are difficult to access. The stretchers are for professional use only. Models available:

- Model 71: Standard basket stretcher
- Model 71-S: Splits in two for storage or backpacking to rescue site

General Guidelines for Use

Medical advice is beyond the parameters of this manual. It is the operators' responsibility to ensure safe practices during use of the product. A minimum of two trained operators is required. Operators must:

- Read and understand this manual.
- Train and practice on proper use of the stretcher following standard emergency patient-handling procedures. Keep training records.
- Inspect the stretcher and confirm it is in good condition.
- Communicate with one another and use coordinated movements to maintain control of the stretcher.
- Stay with the patient at all times.
- Use additional trained help when working with heavy loads (patient plus equipment) and during rescue situations as needed.
- Always use patient restraints, following the instructions in the restraint users' manual. Rescue situations vary and may require a restraint system different from the one supplied with the stretcher. Follow local service's protocols and government requirements.
- If ropes, winches, ladders, lifting bridles ("spiders") or other special equipment will be used, the procedure is considered high-angle or technical in nature and must be carried out only by specially-trained rescue personnel.

Disinfecting and Cleaning

Remove the pad and restraints from the stretcher before cleaning and disinfecting. Refer to the restraints users' manual for instructions.

Wipe all surfaces with disinfectant. Follow the disinfectant manufacturer's instructions for application method and contact time.

Hand clean with warm water and a mild detergent. Rinse with warm, clear water. Dry with a towel or allow it to air-dry.

Inspection Checklist

- Are all components present and securely attached?
- Is the stretcher and pad free of punctures, cracks, deep gouges, and cuts?
- Inspect for discoloration of the orange HPDE material that could indicate wear or damage.
- Is the stretcher frame free of bends and cracks?
- Are restraint webbing and buckles free of visible damage and operate properly?
- **Model 71-S:** Are the center plate, connector pins, and splice pins present and in good condition, and do the stretcher halves attach and detach properly?

Adverse Event Notice

In the event of an adverse event or serious incident related to the use of this device, the end user/operator must report the incident to Ferno at 1-877-733-0911 or via email at tscoordinator@ferno.com. If the incident occurred outside of the U.S., report it to Ferno's Authorized Representative for that area.

Disposal

Prior to disposal, clean and disinfect the product according to instructions in this manual. Recycle or dispose of the product following local protocols and regulations. Do not dispose of the product in municipal waste.

WARNING

Professional use by trained operators only. Use the product only for its intended purpose.

Improper use, operation, or maintenance can cause injury. Use, operate, and maintain only as instructed.

An unattended or unrestrained patient can be injured. Always stay with the patient and use patient restraints.

Improper maintenance can cause injury. Maintain the product only as described in this manual.

Improper parts, modifications, or items not designed for the product can cause injury. Use only Ferno-approved parts and service. Attach only approved items and do not modify the product.

NOTICE

Personnel using the stretcher in technical rescue situations are responsible for knowing the techniques involved and for selecting the stretcher, patient restraint system and other equipment best suited to each rescue situation.

The stretcher is constructed of high-density polyethylene (HPDE). High temperatures and/or direct sunlight may affect the performance and service life of the stretcher. Store the stretcher in a cool, dry area away from direct sunlight.

Do not use disinfectants or cleaners containing phenolics or iodines. If bleach is used, rinse with plenty of clear water to remove bleach residue.

Do not use abrasive materials to clean the stretcher.

Do not immerse restraint metal buckles, slide bars, and clips in liquids or by washing-machine action.

Symbol	Definition	Symbol	Definition
	Follow training protocol		Use proper restraints
	Rope tag lines		Load capacity
	Disinfecting		Product weight
	Cleaning		

Ferno Customer Relations

For ordering assistance or general information, contact your Ferno distributor or Ferno Customer Relations.

Information www.ferno.com
 Authorized Service ferno.com/service-and-support

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