



**Use the board to transfer a load between adjoining surfaces such as a bed and a transport cot or a transport cot and a table.**

- **Where locks are present, lock the wheels on movable equipment used in the transfer.**
- **Always SLIDE the loaded board from one surface to another.**
- **Never lift the loaded board.**
- **Transfer only the weight you can safely move. Use additional help when needed.**

**Important**

Lifting the board may cause its load to shift or fall. SLIDE the board; do not lift it during transfer.



Ferno-Washington, Inc.  
70 Weil Way  
Wilmington, Ohio 45177-9371, U.S.A.  
1.937.382.1451



FERNO S.r.l.  
via B. Zallone, 26  
40066 Pieve di Cento BO  
Italy (+39) 0516860028



Ferno (UK) Ltd.  
Ferno House, Stubs Beck Lane  
Cleckheaton, West Yorkshire, BD19 4TZ  
+44 (0) 1274 851999