AB GERMA®

## BRUKSANVISNING DIRECTIONS FOR USE

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### 1. Warning and caution statements

#### CAUTION

- Avoid handling the mattress next to sharp objects.
- Avoid handling the mattress next to open fire.
- On a current basis check that the mattress has no cracks.

### 2. Intended use

The **Transfer Mattress** (Stretcher/Transfer Mattress) is intended to be used as a comfortable mattress on the stretcher during transportation. It is also a device intended to facilitate the transfer of the body from one stretcher to another or/and lifting and carry of a body.

The mattress can be carried by two persons for a shorter distance, and a longer distance if two or more persons use additional carrying belts. You can pull the mattress on the ground or down stair cases by the handles or by connecting a supporting set Step by Step.

The mattress consists of an outer shell of airtight, fully welded, material that is filled with a foam inlay and is fitted with handles on two sides for carrying or pulling.

### 3. Specifications

The Mattress/Transfer Mattress is tested in accordance with EN 1021-1  
The Mattress /Transfer Mattress is in conformity with Council Directive 93/42/EEC concerning Medical Devices and EN 1865.

**Temperature range:** -30°C to +70°C

#### Parts/Materials

Outer shell: PVC coated fabric (flame retardant)  
Filling material: Foam of polyester

Dimensions	Length (cm)	Width (cm)	Height (cm)	Weight (kg)	N °of handles
<b>Transfer Mattress 12</b>	195	50	3,0	3,8.	12

### 4. Description

The Transfer Mattress is equipped with 6 handles at each side.

The mattress consists of an outer cover of hard-wearing PVC coated polyester weaved canvass with one layer foam filling.

### 5. Operating instructions

1. Place the body in the desired position on the Transfer Mattress.
2. Lift and/or slide the mattress and put the mattress on the stretcher.
3. Close the belts of the stretcher according to instructions.
4. Check that the unit is stable and safe for the body.
5. The body is ready to be transported
6. The Transfer Mattress can be used as a stretcher mattress. The dimensions allow the mattress to fit any commonly sold stretcher.

## 6. Cleaning and Disinfecting

At a regular basis, the Transfer mattress should be wiped with a moist cloth and dried, if dirty, the mattress may be washed with soapy water and rinsed with clean water.

Allow the mattress to dry before storage.

If the transfer mattress needs disinfection first clean the mattress carefully and let it dry. Pour a solution containing 70 % alcohol or the disinfectant called LYSETOL AF on a clean cloth. Smear the transfer mattress with the cloth.

## 7. Repair of Transfer Mattress

If damage occurs repair as follows:

- a) Find a patch in a suitable size and put glue on both the patch and the damaged spot on the transfer mattress.
- b) Wait for the glue to become dust dry (approx.2 min.) and place the patch on the damaged spot.
- c) Press firmly the whole area of the patch, so that there is a good contact and all air is pressed out under the patch.

The repair will receive full strength after 24 hours.

## 8. Storage

The Transfer Mattress is often positioned on top of the stretcher with a covering blanket.

## 9. Ordering information

Art no: 271 800 801 Transfer Mattress 12 – Black

## 10. Spare parts

Art no: (271 800 801) Repair kit – black model

## 11. Accessories

Transfer mattress offers you the possibility to be equipped with extra carrying handle system to support sliding of the mattress on the floor and/or down stair cases.

Art no: 151 000 000 17

Note that all type of restraint system is not crash tested together with our system and length and shape could be out of capability to deliver full safety.

## 12. Maintenance

When the mattress is on the way to be worn on the surface and cracks are starting to show shall the mattress be taken out of duty and destroyed due to the fact that soil and bacteria can be spread and cannot be cleaned with normal process. The mattress shall be checked regularly for any damages.