

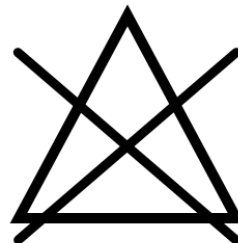
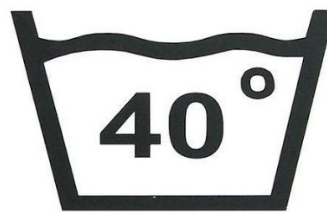
WASHING INSTRUCTIONS HEALTHCARE BAGS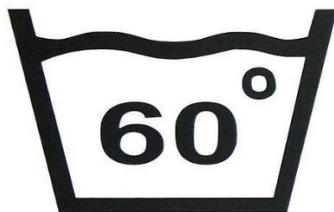

ENGLISH-SWEDISH-NORWEIGIAN-FINNISH- DANISH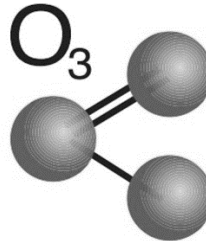

WASHING INSTRUCTIONS HEALTHCARE BAGS

General washing instructions

- Hand washing with soap or mild detergent is always recommended as the lifespan of the medical bag is significantly extended regardless of material.
- When disinfecting with alcohol, hydrogen peroxide or other strong cleaning agents, the material may be bleached. It is of the utmost importance that the medical bag is rinsed very thoroughly after using disinfectant. Remaining residues of strong detergents will damage the medical bag.
- Fabric / nylon bags can be machine washed on a mild program at a maximum of 40 ° C. Note that bags with reinforcements that can be made of wood / paper or similar must be removed. Transparent windows cannot be machine washed. Do not tumble afterwards. Feel free to use a net bag when washing in a machine to protect the medical bag.
- Some simple medical bags can be washed at 60 ° C.
- Zippers can be cleaned with a small brush. They may need to be lubricated afterwards.
- Some bags are impregnated for liquid-repellent effect. The impregnation needs to be redone after washing and at regular intervals to maintain effect.
- Some bags are treated with antibacterial properties. This property decreases over time and with repeated washing.

Ozone cleaning

- Starting with ozone cleaning to neutralize all viruses and bacteria before washing with water and detergent is the most effective method of cleaning.

Important information

- Cooler bags that are often used in sports contexts have corrosive properties and should a cooler bag leak into the medical bag, rinse immediately.
- Always allow medical bags to air dry.
- Avoid detergents with bleach.
- Rinse thoroughly after using detergent.
- Repeated machine washing shortens the life of the medical bag.

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